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System Owner

Thank you for purchasing the Universal Infrared Forehead Thermometer. Please read this instruction manual first so you can use this thermometer safely and correctly.

Always check our website for the latest version of this product's user instructions.

IMPORTANT SAFETY INSTRUCTIONS, READ BEFORE USE

- The following basic safety precautions should always be taken:
1. Close supervision is necessary when the thermometer is used by, on, or near children, handicapped persons or invalids.

BEFORE YOU BEGIN

Cautions and Warnings:

- As with any thermometer, proper technique is crucial to obtaining accurate temperature readings. Please read this manual thoroughly and carefully before using.
Always operate the thermometer in an operating temperature range 16°C to 40°C (60.8°F to 104°F), and relative humidity less than 85%.

Restrictions of use:

This thermometer is clinically proven to produce accurate temperature measurements. However, please be advised that the accuracy cannot be ensured when the thermometer is not clean.

Intended use:

Universal Infrared Forehead Thermometer is intended for the intermittent measurement and monitoring of human body temperature from forehead measurement at home.

- 1. If the reading is surprisingly low,
2. For new-born infants up to 100 days old,
3. For children under three years of age who have a weakened immune system or who react unusually in the presence or absence of fever.

How does it work:

The thermometer measures the infrared heat generated by the surface of the skin over the vessel and its surrounding tissue.

Note: The thermometer does not emit any infrared signal.

DETAILED INFORMATION

About Normal Body Temperature & Fever:

- The temperature in the forehead and temple area differs from the internal temperature, which is taken orally or rectally.
Vasoconstriction, an effect which constricts the blood vessels and cools the skin, can occur during the early stages of a fever.

MAINTENANCE

Care & Cleaning

- The probe is not waterproof. Please wipe with a clean and dry cotton swab to clean the probe on the inside.
The body of the thermometer is not water-resistant.

Environment: (discarding this product)

- When discarding this product, please return it to your retailer; they'll ensure environment-friendly processing.
Exhausted batteries should be brought to your local collection point for small chemical waste.

DECLARATION OF CONFORMITY

The declaration of conformity is available at: http://DOC.hesdo.com/BC-37-DOC.pdf

WARRANTY

For the product you have a warranty of 24 months from the date of purchase. During this period, we guarantee the free repair of defects caused by material and workmanship errors.

The warranty becomes null and void: In case of improper use, incorrect connections, leaking and/or incorrectly installed batteries (if applicable), use of unoriginal parts or accessories, negligence and in case of defects caused by moisture, fire, flooding, lightning and natural disasters.

Service Help icons, WWW.ALECTO.NL SERVICE@ALECTO.NL, Alecto logo, Commaxx B.V., Wiebachstraat 37, 6466 NG Kerkrade, The Netherlands